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CONTENTS

	Page
1. Checking the wheelchair on delivery	14
2. Description of the Snellius^{Comfort}	14
3. Maintaining the Snellius^{Comfort}	19
4. Guarantee	20
Checklist	21

1. CHECKING THE WHEELCHAIR ON DELIVERY

Check whether the wheelchair delivered corresponds to the packing note. If your wheelchair is incomplete or damaged on delivery, please contact your dealer. Your dealer must notify the manufacturer or supplier of the incomplete delivery or damage within two weeks.

2. DESCRIPTION OF THE SNELLIUS^{COMFORT}

The Snellius^{Comfort} seat element is a patented product and complies with the Essential Requirements for Medical Devices Directive 93/42/EEC. The Snellius^{Comfort} seat element has a CE mark.

2.1 FEATURES

The Snellius^{Comfort} is a completely freely suspended seat element. The stretch neoprene cover has a dynamic effect and permits minor posture adjustments in order to avoid localised tiredness, with no loss of support where it is needed. The result is a comfortable, safe feeling for the user.

2.2 TARGET GROUP

Posture adjustments are not a top priority for passive wheelchair users with a lower level of fitness. Much more important is providing maximum support and sitting comfort in the foetal position they prefer. The dynamic Snellius^{Comfort} seat element has been specially developed for this group of bedridden users based on the PDL principle.

2.3 DIMENSIONS

	seat width	seat depth	backrest height
No1	34 cm small	40 cm small	55 cm
No2	34 cm	48 cm medium	55 cm
No3	40 cm	40 cm small	55 cm
No4	40 cm medium	48 cm medium	55 cm one size
No5	40 cm	56 cm large	55 cm
No6	46 cm	48 cm medium	55 cm
No7	46 cm large	56 cm large	55 cm

BECAUSE OF THE FLEXIBLE NATURE OF THE SNELLIUS^{COMFORT}, ALL DIMENSIONS ARE APPROXIMATE.

2.4 ADJUSTMENT OPTIONS

■ Backrest adjustment

The Snellius^{Comfort} has an adjustable backrest. To adjust the angle of the backrest, unzip the neoprene cover at the side and unscrew the cross-shaped block for fastening the back of the Snellius^{Comfort} and for the backrest angle adjustment (and the seat depth adjustment if applicable) on the frame.

Use a size 8 Allen key to loosen and tighten the toothed hinge.

N.B.

- ONLY ADJUST THE BACKREST WHEN THERE IS NO-ONE SITTING IN THE CHAIR.
- MAKE SURE YOU DO NOT COMPLETELY REMOVE THE BOLT IN THE TOOTHED HINGE.
- WHEN ADJUSTING THE BACKREST ANGLE, MAKE SURE THAT THERE IS A GAP OF AT LEAST 10 CM BETWEEN THE SNELLIUS SEAT ELEMENT AND THE CANTO SUBFRAME.

2.5 FOAM ELEMENT

There is a specially moulded foam element in the seat surface which distributes the pressure and supports the body over a large area. At the front of the seat there are leg recesses which also increase the contact surface area.

2.6 LEG SUPPORT

Different types of leg supports are available. Your dealer can advise you which to use.

2.7 ARMRESTS

■ Flex arm support

The Flex arm support is secured to the seat element with a zip at shoulder height.

There are various types of armrests available in addition to the Flex arm support. Your dealer can advise you which to use.

2.8 HEADREST

Depending on the position of the head, you can choose between different types of headrests

■ Snellius headrest

This is a flexible padded headrest.

■ Snellius headrest with Edison lever

This is a flexible headrest which can be very easily fixed in many different positions. If the head moves from a central position and needs more support at the side, the correct support can be provided with one movement of the hand.

THERE IS A PLASTIC HEADREST ATTACHMENT ON THE SEAT ELEMENT FOR FITTING THE HEADREST.

2.9 COVERS

There are different covers available for the Snellius^{Comfort} seat element.

- Terry cover with openings for the seat element. The terry cover is made of 80% cotton for added absorbency.
- Comfort cover with openings for the seat element. The Comfort cover is made of Dacron with a terry top layer. The Dacron optimises contact with the body, giving the user an extremely comfortable feeling. The terry top layer is made of 80% cotton for added absorbency.
- Terry cover for the Flex arm support and headrests.

Attaching the covers to the seat element

There are two tapes on the inside of the cover which are threaded through the seat to the outside in the two corners of the seat. There are also two stretch tapes on the edge of the cover along the seat surface which go round the outside instead of through the seat. All tapes must be tied to the tiltable top part of the frame.

Attaching the covers to the Flex arm support

Once the terry covers have been fitted over the Flex arm support, you can attach them to the back tube to which the push handles are attached.

3. MAINTAINING THE SNELLIUS^{COMFORT}

3.1 CLEANING THE NEOPRENE COVER

The neoprene cover is easy to clean with a damp cloth and an all-purpose cleaning agent that does not contain any aggressive substances and that is suitable for textiles and plastics.

3.2 WASHING THE COVERS

- The terry cover and the Comfort cover for the seat element can be washed at up to 60°.
- The terry covers for the headrest and the Flex arm support can be washed at up to 60°.
- The Flex arm support can be washed at up to 60°.

Tip: Use liquid laundry detergent.

N.B.

- THE FLEX ARM SUPPORTS MUST BE COMPLETELY EMPTIED.
- READ THE LAUNDRY LABEL ON THE COVERS FOR CARE INSTRUCTIONS.

3.3 FLEX ARM SUPPORT

- Granule size : The Flex arm support is filled with 2.5mm polystyrene granules.
- Filling level: The Flex arm supports are filled to a specific standard. No liability can be accepted if their contents are changed by third parties without consulting the supplier or manufacturer.
- Zips: The zips on the Flex arm support must be completely closed.

4. GUARANTEE

The manufacturer guarantees the frame of the Snellius^{Comfort} against frame breakage for a period of one year. The manufacturer also guarantees the quality of the sewn seams on the outer cover and the granule cushions for one year.

The guarantee covers manufacturing defects only and excludes: abnormal or improper use, negligence, overloading or accidents. This guarantee supersedes any other guarantee laid down in law or verbally notified, with the exception of any written guarantee provided by the manufacturer.

Guarantees are only valid within Europe.

Defects must be reported to the manufacturer or supplier within 4 weeks.

Please contact your dealer if components need replacing.

Checklist

✓ SITTING POSITION

Seat the user as far back in the seat element as possible so as to prevent them from slumping back in the chair.

✓ BACKREST ANGLE

The wrong backrest angle can affect the sliding force which can result in an increased risk of decubitus. N.B.: Always adjust the backrest angle when there is no-one sitting in the chair. There must be a gap of at least 10 cm between the Snellius^{Comfort} seat element and the frame.

✓ TILTING

Altering the seat angle regularly reduces the risk of decubitus. The seat angle can be altered using the tilt adjustment on the frame.

✓ TRANSFER

Transfer the user in and out of the chair with a passive lift; position the user on the tilted seat element from the side.

✓ LIFT MAT

Make sure there are no folds, edges or fastening clips of the lift mat between the user and the seat element as they can increase the risk of decubitus. Remove the lift mat after the transfer if possible.

✓ COVERS

The covers must be positioned in such a way that the laundry label is on the top. Secure the tapes to the tiltable part of the frame. When fitting the covers, make sure they are as smooth as possible with no folds.

✓ CLEANING

The neoprene cover is easy to clean with a damp cloth and an all-purpose cleaning agent that does not contain any aggressive substances and that is suitable for textiles and plastics.

✓ WASHING INSTRUCTIONS

The Comfort cover, the terry covers and the Flex arm support can be washed at up to 60°. N.B.: The Flex arm supports must be completely emptied before washing. Tip: Use liquid laundry detergent.

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